

# 3.

## LPG ENDERMOLOGIE® COMBINED

---



# 3. WHY ?

- TO PROPOSE A **GLOBAL** AND **SAFE** APPROACH
- FOR THOSE WHO ARE SCHEDULED FOR **HEATING\*** OR **FREEZING\*** PROCEDURES OR **LIPOSUCTION\*** OR **INJECTIONS ...**
- TO **FACILITATE** INTERVENTIONS (SURGERY, INJECTIONS) AND/OR ADD A PROCEDURE WHICH IS **VERY PLEASANT**
- TO **PROVIDE BETTER INTEGRATION** OF THE INJECTED PRODUCTS (MESOTHERAPY, FILLERS)
- TO **CONTROL HEALING** PROCESS WHEN NEEDED
- TO **REDUCE** AND **REPAIR** POTENTIAL **SIDE EFFECTS** (INFLAMMATION, TINGLING, PAIN, BRUISING, FIBROSIS, OEDEMA, NUMBNESS)
- TO **OPTIMISE FINAL AESTHETIC OUTCOME** AND **MAINTAIN RESULTS**
- TO **INCREASE** PATIENT **SATISFACTION** AND **LOYALTY**

# 3. BODY COMBINATIONS

## CELLULITE



- AWT,
- IR
- RF
- CARBOXY
- MESO
- LLLT



- ABDO'PLASTY
- RF
- LASERS

## LOCALISED FAT




- LIPOSUCTION
- CRYOLIPOLYSIS



## SAGGING SKIN

# 3. FACE COMBINATIONS

## SAGGING SKIN

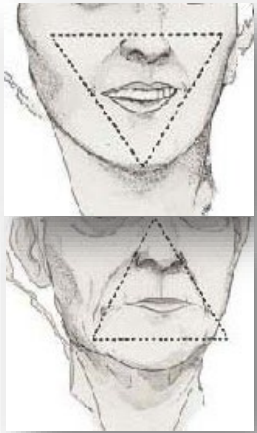

- RF
- SURGERY
- THREADS
- US



- FILLERS




LOSS OF VOLUME

- LASERS
- PEELS
- LED, PRP,
- CARBOXY
- MESO
- TOXINS



TEXTURAL CHANGES

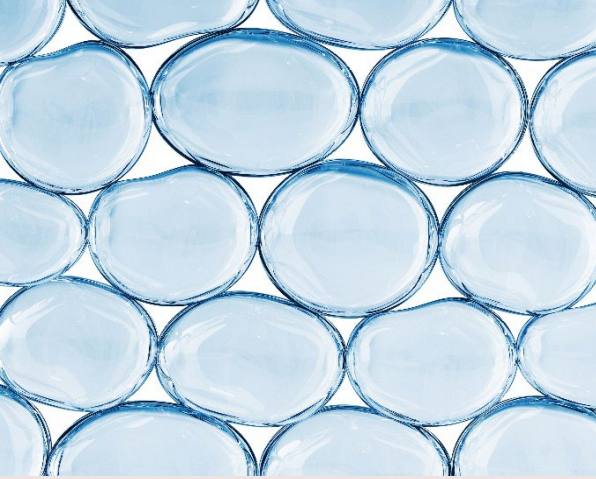

4.

PRE - CO - MA - RE

---

# 4. PRE - CO - MA - RE



ENDERMOLOGIE®  
COMBINED

## **PREPARE**

CUTANEOUS & SUBCUTANEOUS TISSUE  
RESTORE CELLULAR ACTIVITY

1

STIMULATE MICRO-CIRCULATION, MOBILISE INTERSTITIAL FLUIDS, SOFTEN TISSUES

## **COMBINE**

TREAT THE INDICATIONS

2

BODY: CELLULITE, LOCALISED AND RESISTANT FAT, SAGGING SKIN  
FACE: LOSS OF OVAL, WIDENING OF FACIAL CURVES, CHANGES IN SKIN QUALITY

## **MAINTAIN**

INCREASE PATIENT LOYALTY

3

MOBILISE FLUIDS, RELIEVE PAIN, PROMOTE NATURAL LIPOLYSIS\*. STIMULATE COLLAGEN AND ELASTIN PRODUCTION\*\*

## **REPAIR** (RESCUE DEVICE)

**TREAT** POTENTIAL SIDE EFFECTS

0

TO TREAT POSSIBLE OEDEMA, FIBROSIS, SCARS, SKIN SURFACE & CONTOUR IRREGULARITIES

*\* Prof. M. Lafontan study - \*\* Ph. Humbert study*

# 4. PRE – CO – MA – RE

## PREPARE



STIMULATE MICRO-CIRCULATION, MOBILISE INTERSTITIAL FLUIDS, SOFTEN TISSUES

**HOW:** 2-4 SESSIONS OF LPG® ENDERMOLOGIE®

**WHEN:** THE WEEKS BEFORE COMBINE -> 2 TO 3 TIMES/WEEK

**WHICH LPG PROTOCOLS:** A FULL RANGE OF TREATMENTS ACCORDING TO TARGETED AREAS



FACE




BODY



# REVIVE CELLULAR ACTIVITY BEFORE ANY TREATMENT OR SURGERY ...

STIMULATE THE CONNECTIVE  
TISSUE  
BRING BACK ITS REACTIVITY

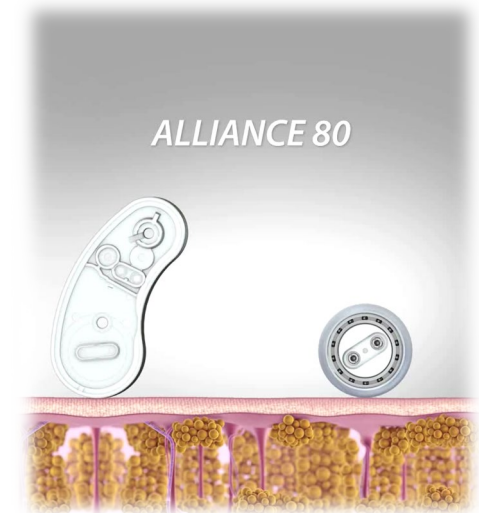

PLANTING SEEDS IN  
AN ARID GROUND IS  
NEVER FRUITFUL!



# 4. PRE – CO – MA – RE

## COMBINE 2


**BODY:** CELLULITE, LOCALISED AND RESISTANT FAT, SAGGING SKIN

**FACE:** LOSS OF OVAL, WIDENING OF FACIAL CURVES, CHANGES IN SKIN QUALITY (WRINKLES, FINE LINES, SPOTS, ROSACEA)

**HOW:** WITH THE DIFFERENT AESTHETIC PROCEDURES AVAILABLE IN THE MEDICAL CENTER, ACCORDING TO PATIENT'S EXPECTATIONS (RF, CRYO, INJECTIONS, PEELS..).

**WHEN:** DAY OF PROCEDURE

**WHICH LPG PROTOCOLS:** PRE-INTERVENTION 8'

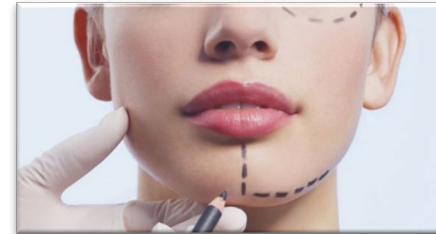

# 4. PRE – CO – MA – RE

## MAINTAIN

3

MOBILISE FLUIDS, RELIEVE PAIN, PROMOTE NATURAL LIPOLYSIS. STIMULATE COLLAGEN AND ELASTIN PRODUCTION

**HOW:** SESSIONS OF LPG® ENDERMOLOGIE **AS YOU WISH!**

**WHEN:** 3 OR 14 DAYS AFTER COMBINE (DEPENDING ON PROCEDURES)

**WHICH LPG PROTOCOLS:** A FULL RANGE OF TREATMENTS ACCORDING TO PATIENT EXPECTATIONS AND INDICATIONS.

BODY: 10 AREAS (5 TO 20 MN TREATMENT)




FACE: 9 AREAS (15 TO 35 MN TREATMENT)



# 4. PRE – CO – MA – RE

**REPAIR**

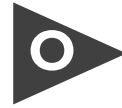




TREAT POSSIBLE SIDE EFFECTS:  
HEMATOMA, OEDEMA, FIBROSIS, SCARS, SKIN  
SURFACE IRREGULARITIES

## ALLY DEVICE – SIDE EFFECTS

**HOW:** 4-10 SESSIONS OF LPG® ENDERMOLOGIE

**WHEN:** ASAP (DOCTOR'S DIAGNOSTIC)

**WHICH TREATMENT:** ACCORDING TO INDICATIONS (SCARS, FIBROSIS, OEDEMA...)

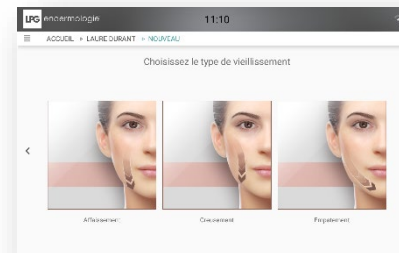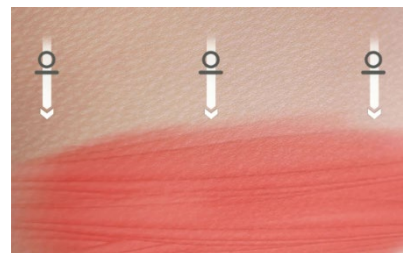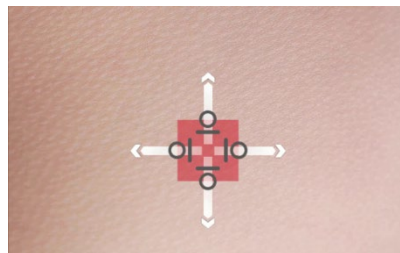



5.

## COMBINED FACE PROTOCOLS

---

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FILLER\*

Before injections: skin preparation + skin circulation tone and firmness enhanced.

After: ensure long lasting results. Stimulation around the injections points to stimulate the elasticity and the replenishment of the skin.

### INDICATIONS:

NASOLABIAL FOLD  
SKIN TEXTURE



BEFORE



AFTER

NUMBER OF LPG SESSIONS: 6 SESSIONS  
DURATION OF THE PROTOCOL: 4 WEEKS  
AFTER THE PROTOCOL: 1 SESSION/MONTH

### PREPARE

WEEK -1

2 LPG sessions

### COMBINE

DAY OF PROCEDURE

10' LPG + FILLER\*

### MAINTAIN

WEEK 2 -3 -4

2 LPG sessions

UP TO WEEK 5

1/MONTH  
LPG SESSION

### LPG TREATMENT:

YOU DONT DRAIN A DEEP FOLD  
35 MIN FULL FACE REPLUMPING

### REPLUMPING LOCALLY

20' «TONING PROTOCOL»  
AROUND

FULL FACE SESSION  
ACCORDING EXPECTATION  
ALWAYS AROUND FILLERS

\*HYALURONIC ACID

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FILLER\*

### INDICATIONS:

LOSS OF VOLUME  
CUTANEOUS SAGGING - DULL SKIN



BEFORE

AFTER

NUMBER OF LPG SESSIONS: 5 SESSIONS  
DURATION OF THE PROTOCOL: 3 WEEKS  
MAINTENANCE ONCE A MONTH RECOMMENDED, AROUND FILLER »  
PROTOCOL REPLUMPING » OR « TONING »

### PREPARE

WEEK -1 AND -2

2 LPG SESSION  
PER WEEKS

### LPG TREATMENT:

30' «REPLUMPLING F.  
ENDERMOLOGIE»

### COMBINE

DAY OF PROCEDURE

15' LPG + FILLER\*

15' « PRE-INTERVENTION » ON  
THE AREA


*\*HYALURONIC ACID*

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FILLER\*

### INDICATIONS:

NASOLABIAL FOLD  
LOSS OF SKIN TONICITY

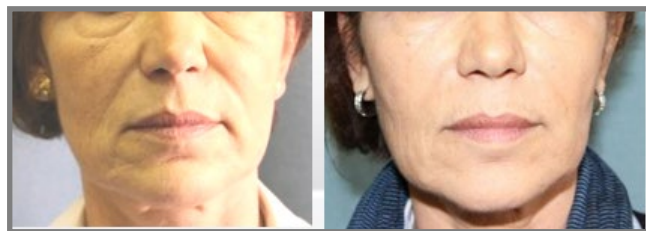

BEFORE

AFTER

NUMBER OF LPG MS SESSIONS: 8 SESSIONS  
ONCE A MONTH ALMOST FOR 5 MORE  
MONTH

	WEEK -1	
<b>PREPARE</b>	2 LPG SESSION 20'	<b>LPG MS TREATMENT:</b> 20' «F. ENDERMOLOGIE DETOX»
<b>COMBINE</b>	DAY OF PROCEDURE FILLER* + 8' LPG MS	<b>LPG MS TREATMENT:</b> PRE : 8' «PRE INTERVENTION » TREATMENT OF LASOLABIAL FOLD POST : 8' « FREE MENU »ON THE AREA
<b>MAINTAIN</b>	1 SESSION PER MONTH LPG MS	<b>LPG MS TREATMENT:</b> 30' «TONING F. ENDERMOLOGIE AROUND THE FILLER

\*HYALURONIC ACID

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + LIPS FILLERS\*

Endermologie® lips session = immediate glow and volume.  
After: maintain the results while working around the fillers.



### PREPARE

WEEK -1 & -2

1 LPG SESSION 20'

### LPG TREATMENT:

Lips LPG session 20'

### COMBINE

DAY OF PROCEDURE

1 LPG SESSION 10'

LPG lips session 10' and fillers

### MAINTAIN

UP TO WEEK 1

1/ month LPG  
SESSION 35'

1 monthly redensifying LPG session full face neck and targeting the oval of the face and lips

NUMBER OF LPG SESSIONS: 3 SESSIONS

*\*HYALURONIC ACID*






# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FILLER\*

\*HYALURONIC ACID

### INDICATIONS:

NASOLABIAL FOLD LOSS OF SKIN TONICITY

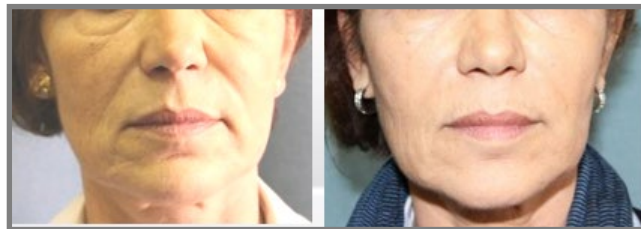

BEFORE

AFTER

NUMBER OF LPG MS SESSIONS: 6 SESSIONS  
ONCE A MONTH FOR 4 MORE MONTH

TO ALLOW A BETTER INTEGRATION OF THE INJECTED PRODUCT

### PREPARE

WEEK -1

2 LPG SESSION 20'

**LPG MS TREATMENT:**  
20' «F. ENDERMOLOGIE  
DETOX»

### COMBINE

DAY OF PROCEDURE

FILLER\* +  
1 LPG SESSION

**LPG MS TREATMENT:**  
5' POST-INJECTIONAL: 1  
IMMEDIATE SESSION AROUND  
THE INJECTION

### MAINTAIN

MONTH 2-3-4

1 SESSION PER  
MONTH LPG MS

**LPG MS TREATMENT:**  
30' TO BOOST WATER  
FIXATION AND RESTART  
HYALURONIC ACID SYNTHESIS

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + BOTULIN TOXIN

Endermologie® prepares skin suppleness prior to injection, reduces fine line and gives more body to the skin

**INDICATIONS:** WRINKLES FOREHEAD & CROW'S FEET



BEFORE

AFTER

**PREPARE**

WEEK -4 to -1

8 LPG SESSIONS

**LPG MS TREATMENT:**

10' « FOREHEAD F. ENDERMOLOGIE »

10' « EYE WRINKLES »

**COMBINE**

DAY OF PROCEDURE

10' LPG + BT

**LPG MS TREATMENT:**

10 « PRE-INTERVENTION

FOREHEAD AND EYES »

NUMBER OF LPG MS SESSIONS: 9 SESSIONS

DURATION OF THE PROTOCOL: 4 WEEKS

MAINTENANCE ONCE A MONTH FULL FACE

FIRMING OR REPLUMPING 30' TREATMENT DEPENDING TYPE OF FACE

BOTOX AREA CAN BE WORKED ON AS IT REACTIVATE THE TOXIN/ LONG LASTING RESULTS

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + BOTULIN TOXIN

**INDICATIONS:** WRINKLES FOREHEAD & CROW'S FEET



BEFORE



AFTER



**MAINTAIN**

WEEK 1

NO MOBILISATION

**LPG MS TREATMENT:**  
NO MOBILISATION FOR 72 HOURS

WEEK 3-4

AT A DISTANCE  
FROM THE  
INJECTION

**LPG MS TREATMENT:**  
10' PERPENDICULAR TO WRINKLES  
IN THE INJECTED AREA TO PLUMP

NUMBER OF LPG MS SESSIONS: 2 SESSIONS  
DURATION OF THE PROTOCOL : 2 WEEKS  
MAINTENANCE ONCE A MONTH FULL FACE. FIRMING OR  
REPLUMPING 15' TREATMENT

Isabelle Breton



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + SUPERFICIAL PEEL

While peeling will remove the dullness of the skin, endermologie® will give back the radiance by increasing the blood flow. The skin is tone radiant and deeply exfoliated.

### INDICATIONS:

DULL COMPLEXION, LOSS OF SKIN TONICITY, UNEVEN PIGMENTATION



BEFORE

AFTER

NUMBER OF LPG MS SESSIONS: 9 SESSIONS

DURATION OF THE PROTOCOL : 8 WEEKS

	WEEK -1	
<b>PREPARE</b>	2 LPG SESSIONS	<b>LPG TREATMENT:</b> 30' «TONING F. ENDERMOLOGIE»
	DAY OF PROCEDURE	<b>WEEK 2</b> 10' TONING LPG + PEELING
<b>COMBINE</b>	<b>WEEK 3</b> 1 LPG SESSION 30' TONING	<b>WEEK 4</b> 30' TONING LPG + PEELING
	<b>WEEK 5</b> 1 LPG SESSION 30' TONING	<b>WEEK 6</b> 30' TONING LPG + PEELING
	<b>WEEK 7</b> 1 LPG SESSION 30' TONING	<b>WEEK 8</b> 30' TONING LPG + PEELING

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + MICRONEEDLING FACE GLOW

Combining endermologie® and microneedling is combining 2 forces to boost the endogenous collagen and elastin. The hyaluronic acid injected in the derm will retain water and pulp the skin while endermologie® will boost the fibroblast to secrete collagen and elastin. Immediate result on the radiance of the complexion, hydration and wrinkles that will be naturally smoothed.

### INDICATIONS :



BEFORE

AFTER

\*NUMBER OF LPG SESSIONS: 4 SESSIONS  
 \*NUMBER OF MICRONEEDLING SESSIONS: 2 SESSIONS EVERY OTHER WEEK  
 MAINTENANCE 1 LPG SESSION PER MONTH

### PREPARE

#### WEEK -1\*

1 LPG SESSION  
35'

#### LPG TREATMENT:

Full Face Neck area the week before

### COMBINE

#### DAY OF PROCEDURE

NO LPG  
TREATMENT

Micro-needling 30'

### MAINTAIN

#### WEEK 1 -3 -5\*

1 LPG SESSION  
35'

LPG 35' full face neck and decolleté

#### UP TO WEEK 5

1 /MONTH LPG  
SESSION 35'

1 LPG session monthly replenishing (firm) full face neck and decolleté



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + HYDRAFACIAL®

HF provides a deep natural cleaning of the epiderm and boost the endogenous collagen. Combined with LPG®, it gives back a clean radiance skin, balancing the pores activity.  
 endermologie® will immediately restore the skin's radiance and tone while reviving fibroblast activity.



NUMBER OF HYDRAFACIAL SESSIONS: 3 SESSIONS  
 NUMBER OF LPG SESSIONS: 3 SESSIONS  
 DURATION OF THE PROTOCOL : 6 WEEKS

### PROTOCOL : 3 HYDRAFACIAL SESSIONS 3 LPG SESSIONS

	<b>WEEK -1</b>	
<b>PREPARE</b>	NO LPG TREATMENT	<b>LPG TREATMENT:</b>
	<b>DAY OF PROCEDURE</b>	
<b>COMBINE</b>	NO LPG TREATMENT	HYDRAFACIAL 30'
	<b>WEEK 1</b>	
<b>MAINTAIN</b>	1 LPG SESSION 35'	LPG 35' FULL FACE NECK AND DECOLLETÉ
	<b>WEEK 3-5</b>	
	1/WEEK LPG SESSION 10'	COMBINE 10' LPG PER AREA THAT NEED A BOOST OF COLLAGEN PRODUCTION EVERY 2 WEEKS FOR 4 WEEKS

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + HYDRAFACIAL®



NUMBER OF HYDRAFACIAL SESSIONS; 3 SESSIONS  
NUMBER OF LPG SESSIONS: 3 SESSIONS  
DURATION OF THE PROTOCOL : 6 WEEKS

<b>PREPARE</b>	<b>WEEK -1</b>	NO LPG TREATMENT	<b>LPG TREATMENT:</b>
	<b>DAY OF PROCEDURE</b>		
<b>COMBINE</b>		NO LPG TREATMENT	HYDRAFACIAL 30'
	<b>WEEK 1</b>		
<b>MAINTAIN</b>		1 LPG SESSION 35'	LPG 35' FULL FACE NECK AND DECOLLETÉ
	<b>WEEK 3-5</b>	1/WEEK LPG SESSION 10'	COMBINE 10' LPG PER AREA THAT NEED A BOOST OF COLLAGEN EVERY 2 WEEKS FOR 4 WEEKS

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + RADIOFREQUENCY

Radiofrequency comes to tightening the skin while combined with endermologie, it will restore contours to create toned firm, youthful-looking skin.  
This firming double action makes RF and endermologie well-suited to the chin, neck, and jawline.

### INDICATIONS:

LOSS OF SKIN TONICITY  
DULL COMPLEXION



BEFORE

AFTER

### PREPARE

WEEK -1

2 LPG SESSIONS

### LPG MS TREATMENT:

30' « TONING F.  
ENDERMOLOGIE » FIRMING

### COMBINE

DAY OF PROCEDURE

10 LPG MS + RF

### LPG MS TREATMENT:

8' « PRE-INTERVENTION » FIRMING

NUMBER OF LPG MS SESSIONS: 3 SESSIONS

DURATION OF THE PROTOCOL: 2 WEEKS

TO BE DONE EVERY 3 MONTHS



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + RADIOFREQUENCY

RF is well renowned to tone and firm the skin. but you need to wait 2 to 3 weeks in between the sessions. endermologie® is there in between to keep stimulating the endogenous collagen and elastin. Maintain or restore the firmness of the skin while not invasive solution.



**PREPARE**

**COMBINE**

**MAINTAIN**

### PROTOCOL : 2 RF / 3 LPG SESSIONS

#### WEEK -1

1 LPG SESSION  
35'

#### LPG TREATMENT:

LPG 35' full face neck and décolleté

#### DAY OF PROCEDURE

NO LPG  
TREATMENT

Radiofrequency 30'

#### WEEK 1-2

1 LPG SESSION  
35'

LPG 35' full face neck and  
décolleté

#### WEEK 3

NO LPG  
TREATMENT

Radiofrequency 30'

#### UP TO WEEK 3

1/ MONTH LPG  
SESSION 35'

LPG 35' full face neck and  
décolleté

NUMBER OF RF SESSIONS: 2 SESSIONS


NUMBER OF LPG SESSIONS: 3 SESSIONS

DURATION OF THE PROTOCOL: 4 WEEKS (TBD TWICE A YEAR)

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + RADIOFREQUENCY

**INDICATIONS :** RESTORING CONTOURS TO CREATE TONED FIRM YOUTHFUL LOOKING SKIN TARGETING CHIN NECK AND JAW LINE

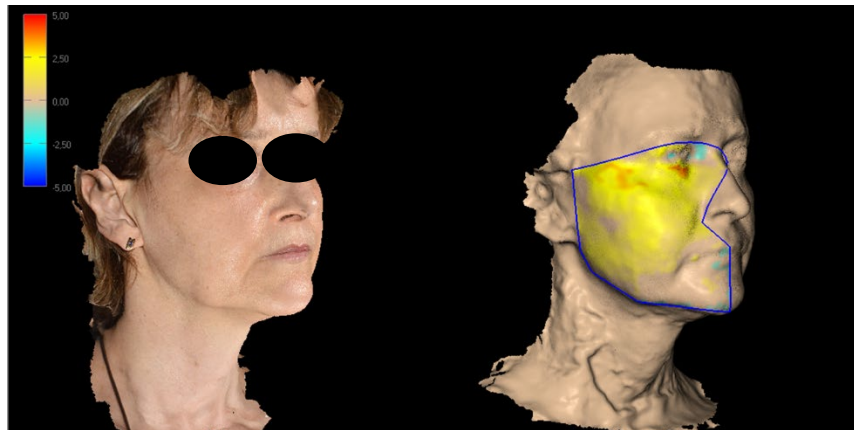

NUMBER OF RF SESSIONS: 5 SESSIONS  
 NUMBER OF LPG SESSIONS: 6 SESSIONS  
 DURATION OF THE PROTOCOL: 6 WEEKS

### PREPARE

#### WEEK -1

1 LPG SESSION 30'

LPG 30' full face

### COMBINE

#### WEEK 1-2-3-4-5

1 LPG SESSION 10' +  
RF

LPG 10' full face

### MAINTAIN

#### UP WEEK 5

2 LPG  
SESSIONS/MONTH

LPG 30' full face


3 MONTHS



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + RADIOFREQUENCY

**INDICATIONS:** SAGGING SKIN

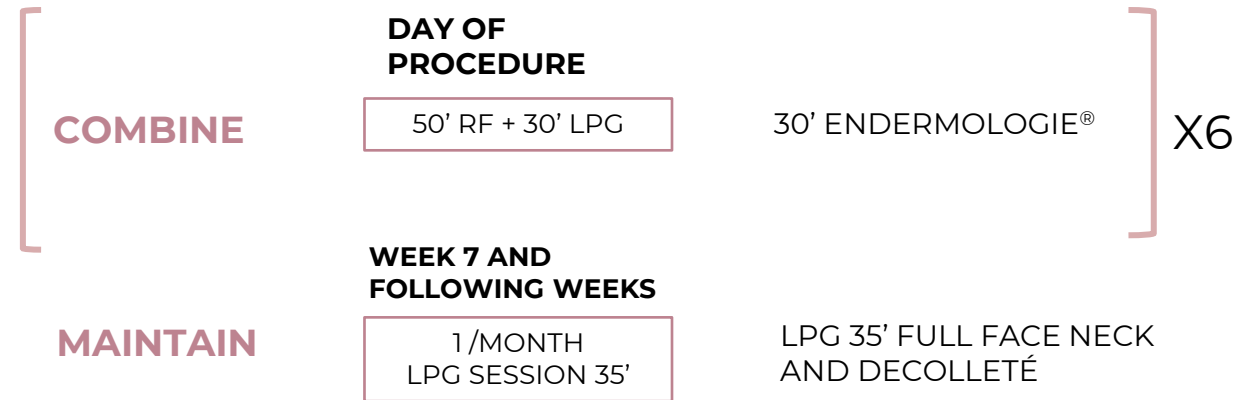

NUMBER OF HYDRAFACIAL SESSIONS: 6 SESSIONS

NUMBER OF LPG SESSIONS: 6 SESSIONS

DURATION OF THE PROTOCOL: 6 WEEKS



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + \*MINI LAL

**INDICATIONS:** DOUBLE CHIN



BEFORE

AFTER

**COMBINE**

**DAY OF PROCEDURE**

NO LPG  
TREATMENT

MINI LAL

**MAINTAIN**

**WEEK 1 to 5**

10 LPG SESSIONS

30' «STREAMLINE »

NUMBER OF LPG S SESSIONS: 12 SESSIONS

DURATION OF THE PROTOCOL : 6 WEEKS

\* Laser Assisted Liposuction

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + THREADS

IF SUSPENSIONS CHEEKBONES BY THREADS

- WORK RECOVERY OF THE ORAL OPENING
- RELAX THE TEMPORAL MUSCLE

**SIDE  
EFFECTS**

### INDICATIONS:

SKIN SURFACE IRREGULARITIES



BEFORE



AFTER

**AFTER SURGERY**

**REPAIR**

**WEEK 1**

5 LPG SESSIONS

LPG Free menu

NUMBER OF LPG S SESSIONS: 5 SESSIONS

DURATION OF THE PROTOCOL: 3 WEEKS

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + THREADS

**SIDE  
EFFECTS**

### INDICATIONS:

OEDEMA OF THE LOWER EYELID



BEFORE

AFTER

**AFTER SURGERY**

**REPAIR**

**WEEK 1**

2 LPG SESSIONS  
10'

LPG 10' Eye endermologie®

NUMBER OF LPG SESSIONS: 2 SESSIONS

DURATION OF THE PROTOCOL: 2 DAYS

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + BLEPHAROPLASTY

**SIDE  
EFFECTS**

### INDICATIONS:

OEDEMA OF THE UPPER EYELID



BEFORE



AFTER

**AFTER SURGERY**

**REPAIR**

**WEEK 1**

6 LPG SESSIONS  
10'

LPG 10' Eye endermologie®

NUMBER OF LPG SESSIONS: 6 SESSIONS

DURATION OF THE PROTOCOL: 3 WEEKS


# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + BLEPHAROPLASTY

**SIDE  
EFFECTS**

### INDICATIONS:

OEDEMA & HEMATOMA

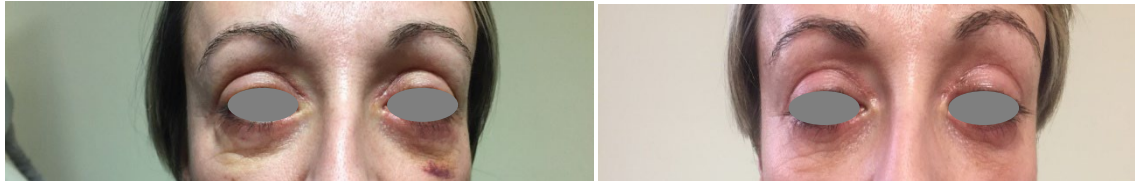

BEFORE

AFTER

**AFTER SURGERY**

**REPAIR**

**WEEK 1**

2 LPG SESSIONS  
10'

LPG 10' Drainage

NUMBER OF LPG MS SESSIONS: 2 SESSION

DURATION OF THE PROTOCOL: 2 WEEKS




# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + BLEPHAROPLASTY

**SIDE EFFECTS**

### INDICATIONS:

HYPERTROPHIC SCARS ON UPPER EYELID AND PIGMENTATION ON LOWER EYELID (ACCUMULATION OF HEMOSIDERIN)

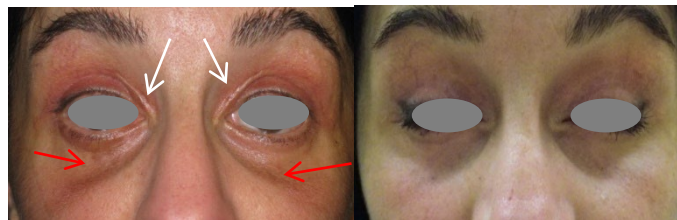

BEFORE

AFTER

NUMBER OF LPG SESSIONS: 22 SESSIONS

DURATION OF THE PROTOCOL: 5 MONTHS

**AFTER SURGERY**


**REPAIR**

**WEEK 1**

22 LPG SESSIONS

LPG Free menu

	SESSION 1 to 3	SESSION 3 to 12	SESSION 13 to 19	SESSION 20 to 22
	2X/WEEK		1X/WEEK	1X/ 3 WEEKS
	25-30 min		25-30 min	25-30 min
<b>SETTINGS FREE MENU*</b>	1) 8 HZ, 30%; I=2; TML20	1) 2 HZ 30% 2) Last min: 16Hz 30%; I=2 TML 20, <u>entire face</u>		<b>Reactivation of lymphatic microcirculation</b> <b>Periorbital deep drainage (following lymphatic pathways)</b> <b>Treatment of orbital scars</b>

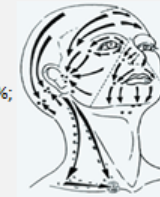

# 5. COMBINED FACE PROTOCOLS ENDERMOLOGIE® + BLEPHAROPLASTY



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + BLEPHAROPLASTY



# 5. COMBINED FACE PROTOCOLS ENDERMOLOGIE® + BLEPHAROPLASTY



# 5. COMBINED FACE PROTOCOLS ENDERMOLOGIE® + BLEPHAROPLASTY



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + MICRONEEDLING\*

\*NCTC + BIOTINE

The LPG technology of mobilisation of the connective tissue by enhancing circulation will prepare the skin and help the needle to get into the skin. The skin will be more responsive and supple. The practitioner will feel easier to work on the skull then. The hair follicle will have his activity boosted by the acceleration of the blood flow locally that it will receive.

### INDICATIONS: HAIR LOSS/ ALOPECIA



BEFORE

AFTER

NUMBER OF LPG SESSIONS: 8 SESSIONS

3 BEFORE - 1 COMBINED - 4 MAINTAIN

+ MAINTENANCE ONCE A MONTH

<b>PREPARE</b>	<b>WEEK -1</b>	3 LPG SESSION 35'	Full Face Neck and Alopecia area
	<b>DAY OF PROCEDURE</b>	1 LPG SESSION 10' + microneedling	10' LPG session on the alopecia area pre-Micro-needling session
<b>COMBINE</b>	<b>WEEK 2 -3 -4-5</b>	4 LPG SESSION 40'	40' LPG session Full face neck and alopecia area every week
	<b>UP TO WEEK 5</b>	1/MONTH LPG SESSION 35'	1 LPG session 35' Full face neck and alopecia area every month
<b>MAINTAIN</b>			



# 5. COMBINED FACE PROTOCOLS


## ENDERMOLOGIE® + INJECTION\*

\*GROWTH FACTORS

Smooth the skin and helps the needle to get to the dermis. This way the absorption of the active ingredients are enhanced. Patient seems to feel less the needle when the LPG stimulation has been done prior, so combining massage and injections is combining forces going same direction.

**Indications:** local hair loss , pregnancy, menopause, chemo, covid, post hair transplants

mesoesthetic pharma group, s.l.

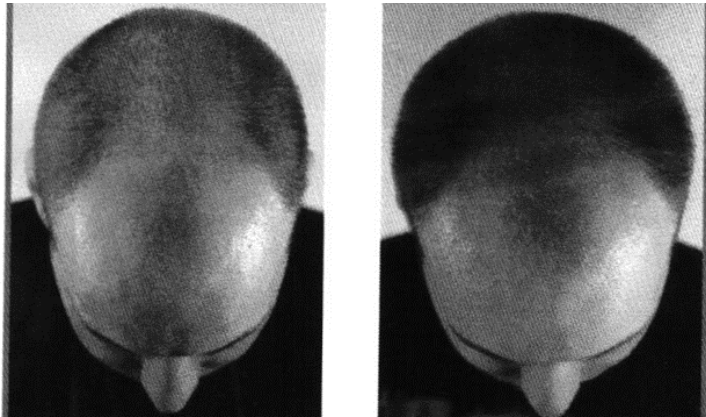

BEFORE

AFTER

NUMBER OF LPG SESSIONS: 12 SESSIONS  
3 BEFORE - 5 COMBINED - 4 MAINTAIN  
+ MAINTENANCE ONCE A MONTH

<b>PREPARE</b>	<b>WEEK 1</b> 3 LPG SESSION 35'	Full Face Neck and Alopecia area
<b>COMBINE</b>	<b>WEEK 2-3-4-5-6</b> 5 LPG SESSION 10' + microneedling	10' LPG session on the alopecia area pre-Injection session
<b>MAINTAIN</b>	<b>FROM WEEK 7 to 10</b> 4 LPG SESSION 40'	40' LPG session Full face neck and alopecia area every week
	<b>UP TO WEEK 10</b> 1/MONTH LPG SESSION 35'	1 LPG session 35' Full face neck and alopecia area every month

# 5. COMBINED FACE PROTOCOLS

## • ENDERMOLOGIE® + FACELIFT

### PRE-SURGERY

- TO REVIVE THE ACTIVITY OF THE FIBROBLAST
- TO FACILITATE THE INTERVENTION WITH ANTI-FIBROSIS ACTION
- ENDERMOLOGIE PREPARE THE SKIN BEFORE BY STIMULATING THE VEINO LYMPHATIC SYSTEM AND THIS WAY DYNAMISE CIRCULATION AND REDUCE THE CHANCE OF BRUISING AND SWELLING.

### POST-SURGERY

- IT IS REALLY INTERESTING TO DRAIN RAPIDLY THE OEDEMA POST SURGERY AND THE BRUISES. THE RESULTS ON THE SCARS WILL BE MUCH BETTER IF WE GET FAST RID OFF THE SWELLING WITH THE ENDERMOLOGIE LYMPHATIC DRAINAGE
- DECREASE OEDEMA (LOCALISED ON THE ENTIRE FACE AND NECK)
- TREATING SCARS
- DECREASE TENSIONS ON SCOM (STERNOCLEIDOMASTOID)
- DEFIBROSING THE ORAL FLOOR IN CASES OF ASSOCIATED LIPOSUCTION
- FIRM THE SKIN
- ACTIVATE WOUND HEALING (IMPROVE MICRO-CIRCULATION) INTEREST +++ IN SMOKING PATIENTS



# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FACE LIFT

### SIDE EFFECTS

**INDICATIONS:** OOEDEMA AND BRUISES POST SURGERY



BEFORE

AFTER

<b>PREPARE</b>	<b>WEEK -1, -2</b> 2 LPG SESSION 35'	LPG 35' SESSION FIRMING AND TONING LPG 35' FULL FACE NECK AND DECOLLETÉ
<b>COMBINE</b>	<b>DAY OF PROCEDURE</b> NO LPG TREATMENT	FACE LIFT
<b>MAINTAIN</b>	<b>WEEK 1 -2-3</b> 2 LPG SESSION 35'	LPG 35' DRAINAGE FULL FACE NECK AND DECOLLETÉ

NUMBER OF LPG SESSIONS: **10 SESSIONS**


DURATION OF THE PROTOCOL: **5 WEEKS**

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FACE LIFT

**SIDE  
EFFECTS**

**INDICATIONS:** POST CENTROFACIAL FACE-LIFT OEDEMA (LOWER EYELID)

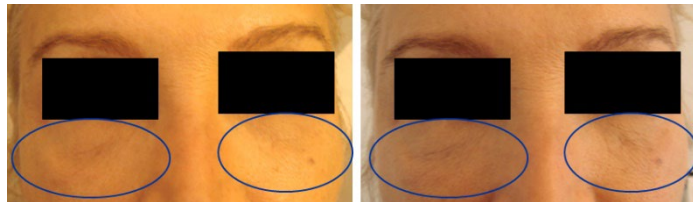

BEFORE

AFTER

**REPAIR**

2 LPG SESSIONS 10'

10' «EYE ENDERMOLOGIE»

NUMBER OF LPG SESSIONS: 1 SESSION

DURATION OF THE PROTOCOL : 1 DAY

# 5. COMBINED FACE PROTOCOLS

## ENDERMOLOGIE® + FACE LIFT

**SIDE  
EFFECTS**

- INDICATIONS :** SCARS POST LIFTING
- STAGE 1 INFLAMMATORY
  - STAGE 2 NON INFLAMMATORY

**PROTOCOL : AFTER SURGERY ON MEDICAL ADVISE AROUND THE INFLAMMATORY SCARS TO BEGIN ON THE NON-INFLAMMATORY SCARS 10'**



**AFTER SURGERY**

**REPAIR**

**WEEK 1**

1 LPG SESSION  
35'

LPG 35' drainage full face neck and décolleté around the scars

**WEEK 2-3-4-5**

2 LPG SESSIONS  
35'

LPG 15' drainage around the scars  
LPG 20' full face neck and décolleté

NUMBER OF LPG SESSIONS: 9 SESSIONS  
DURATION OF THE PROTOCOL: 5 WEEKS  
MAINTENANCE FOR THE SCARS AS NEEDED



6.

## COMBINED FACE RESULTS

---

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + BOTULIN TOXIN

CASE 1: WITHOUT LPG PREPARATION



BEFORE

AFTER

ADHERENT SKIN

INCOMPLETE RESULT

CASE 2: WITH 8 LPG SESSIONS (PRE)



BEFORE

AFTER

ADHERENT SKIN

OPTIMAL RESULT

COURTESY DR MAJANI, ITALY

# 6. COMBINED FACE RESULTS ENDERMOLOGIE® + LIPOSUCTION

CASE 1: WITHOUT LPG POST-LIPO



BEFORE

AFTER

INCOMPLETE RESULT

CASE 2: WITH LPG POST-LIPO (10 SESSIONS)



BEFORE

AFTER

OPTIMAL RESULT

COURTESY DR MAJANI, ITALY

# 6. COMBINED FACE RESULTS ENDERMOLOGIE® + THREAD LIFTS

CASE 1: WITHOUT LPG POST-SURGERY



BEFORE

AFTER

40 DAYS TO HAVE  
FINAL RESULT

CASE 2: WITH LPG POST-SURGERY (3 SESSIONS)



BEFORE

AFTER

10 DAYS TO HAVE  
FINAL RESULT

COURTESY DR MAJANI, ITALY

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + BLEPHAROPLASTY

**SIDE  
EFFECTS**



BEFORE J2



AFTER 2 SESSIONS - J6



# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + BLEPHAROPLASTY

**SIDE  
EFFECTS**



BEFORE J0




AFTER 6 SESSIONS – J21

COURTESY DR PULETTI, ITALY

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + BLEPHAROPLASTY


**SIDE EFFECTS**

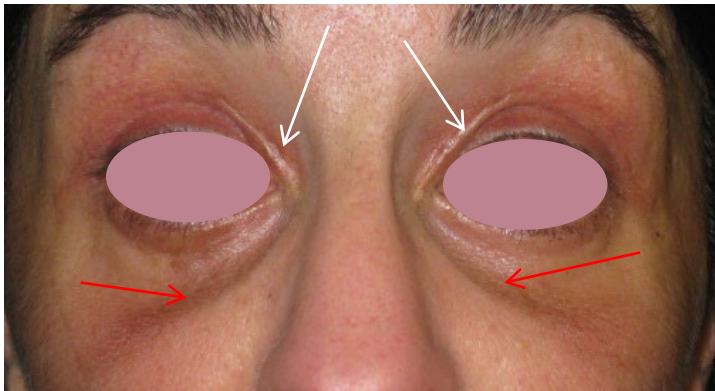

BEFORE



AFTER 10 SESSIONS

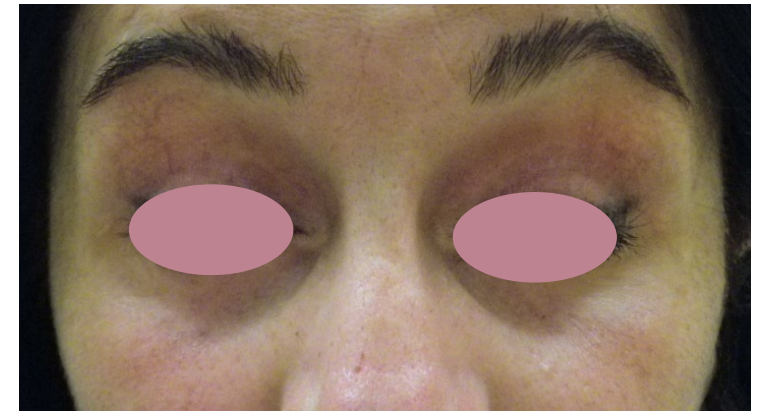

AFTER 20 SESSIONS

COURTESY DR PULETTI, ITALY

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + PEELING\*

\*DERMACEUTICS



BEFORE




AFTER 8 SESSIONS

COURTESY DR HERAUD BARDIN, MARTIGUES

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + PEELING\*

\*DERMACEUTICS

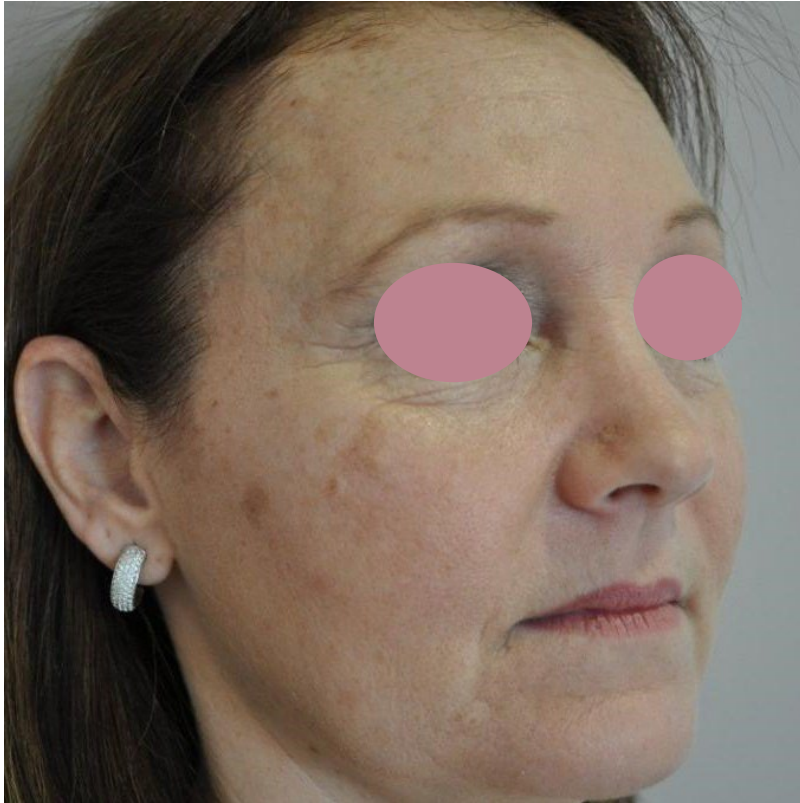

BEFORE



AFTER 8 SESSIONS

COURTESY DR HERAUD BARDIN, MARTIGUES

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + PEELING\*

\*DERMACEUTICS



BEFORE



AFTER 8 SESSIONS

COURTESY DR ACHARD, MARSEILLE

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + PEELING\*

\*DERMACEUTICS



BEFORE



AFTER 8 SESSIONS

COURTESY DR ACHARD, MARSEILLE



# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + PEELING\*

\*Peroxy peel = TCA 33% + H<sub>2</sub>O<sub>2</sub>



BEFORE



AFTER

COURTESY DR ALBERICO, ITALY

# 6. COMBINED FACE RESULTS ENDERMOLOGIE® + PEELING\*

\*Peroxy peel = TCA 33% + H<sub>2</sub>O<sub>2</sub>



COURTESY DR ALBERICO, ITALY




# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + CARBOXYTHERAPY



BEFORE

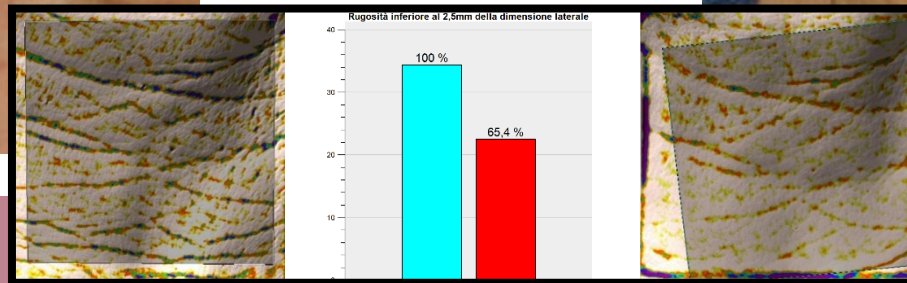


AFTER

COURTESY DR ALBERICO, ITALY

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + CARBOXYTHERAPY

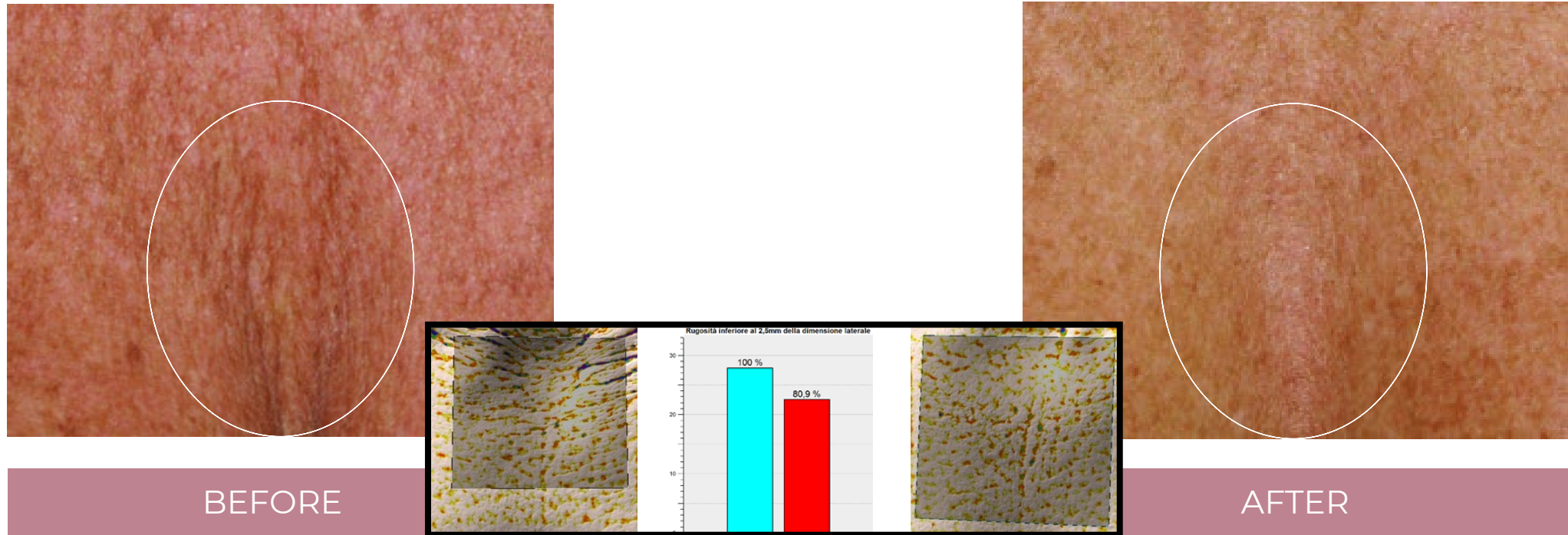

COURTESY DR ALBERICO, ITALY

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + RADIOFREQUENCY



BEFORE



AFTER 3 SESSIONS

COURTESY DR BEILIN, FRANCE

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + LIFTING



BEFORE



AFTER 3 SESSIONS

COURTESY LACLINIC MONTREUX, SWITZERLAND

# 6. COMBINED FACE RESULTS ENDERMOLOGIE® + LIFTING



COURTESY FLORENCE CHARPENTIER, FRANCE


# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + LIFTING



BEFORE

15/01/15

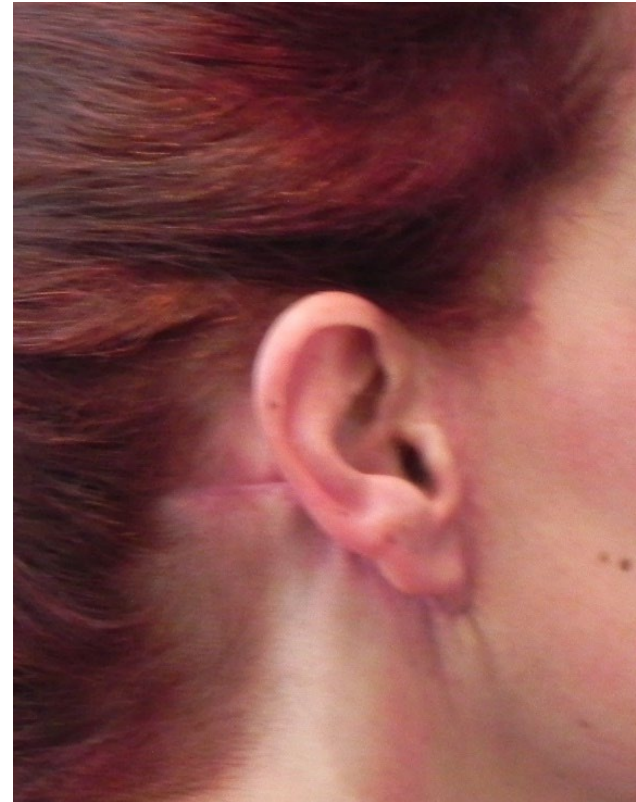

AFTER 10 SESSIONS

23/02/15

COURTESY INGRID LANNO, FRANCE

# 6. COMBINED FACE RESULTS

## ENDERMOLOGIE® + HAIR SURGERY



BEFORE



AFTER 10 SESSIONS (2 PRE + 8 POST)

COURTESY LACLINIC MONTREUX, SWITZERLAND



7.

## COMBINED BODY PROTOCOLS

---





# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + CARBOXYTHERAPY

ON THE DAY OF THE PROCEDURE IT IS INTERESTING TO DO A SESSION OF ENDERMOLOGIE® (CELLULITE) TO RESTART CELLULAR ACTIVITY. THEN IT WILL ALLOW TO DRAIN AND MAINTAIN THE RESULTS

**INDICATIONS :** CELLULITE



NUMBER OF LPG SESSIONS: 10 SESSIONS  
DURATION OF THE PROTOCOL: 10 WEEKS

**COMBINE**

**DAY OF PROCEDURE**

1 LPG SESSION 40'  
+ CARBOXY

40' LPG session  
CELLULITE

X10

**MAINTAIN**

**WEEK 11 & FOLLOWING**

1/MONTH  
LPG SESSION 35'

35' LPG session  
FULL BODY CELLULITE

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + MESOTHERAPY

THIS INVOLVES INJECTING ACTIVE PRODUCTS UNDER THE SKIN USING SINGLE-USE SYRINGES OR WITH AN ELECTRONIC INJECTION GUN. ENDERMOLOGIE WILL BE VERY USEFUL TO OPTIMISE THE RESULTS BY DRAINING AND ESPECIALLY USE LYMPHATIC DRAINAGE

**INDICATIONS:** SMALL ADIPOSITIES



NUMBER OF LPG SESSIONS: 10 SESSIONS  
DURATION OF THE PROTOCOL: 10 WEEKS

**COMBINE**

**DAY OF PROCEDURE**

1 LPG SESSION 40'  
+ MESO

40' LPG session  
LOCALISED FAT

X10

**MAINTAIN**

**WEEK 11 & FOLLOWING**

1/MONTH  
LPG SESSION 35'

35' LPG session  
LOCALISED AREA

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + CRYOLIPOLYSIS



AT **2 MONTHS POST-TREATMENT**, MEAN FAT LAYER REDUCTION WAS **68% GREATER** IN THE **MASSAGE SIDE** THAN IN THE NON-MASSAGE SIDE AS MEASURED AND **4 MONTHS**, MEAN FAT LAYER REDUCTION WAS **44% GREATER** IN THE **MASSAGE SIDE** (CONTROLLED BY ULTRASOUND. Pr GERALD E BOEY 1, JENNIFER L WASILENCHUK - VANCOUVER 2014)

AFTER STOPPING THE VACUUM AND REMOVING THE DEVICE, A MASSAGE OF A FEW MINUTES AIMS TO WARM THE TREATED AREA AND INCREASE THE EFFICIENCY OF THE PROCEDURE (BOEY STUDY-VANCOUVER).


ACCORDING TO **THE LATEST STUDIES**, POST-SESSION MASSAGE :

- **REDUCES INFLAMMATION**
- **OPTIMISES** THE RESULT.

AT A DISTANCE FROM THE TREATMENT, IT IS GOOD TO **CONTINUE** THE **MASSAGE** ESPECIALLY IF YOU HAVE A STRONG **INFLAMMATORY** REACTION


# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + CRYOLIPOLYSIS

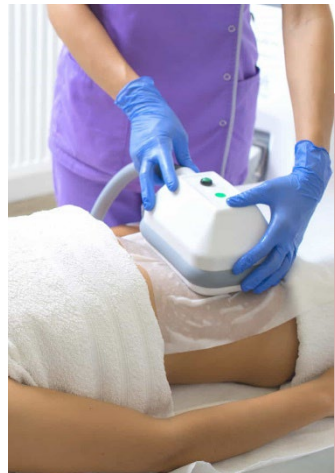

### CRYOLIPOLYSIS

- EFFECT ON LOCALISED FAT
- NO EFFECT ON SAGGING SKIN
- NO EFFECT ON CELLULITE
- NO EFFECT ON AQUEOUS OR FIBROUS FAT
- NO IMMEDIATE VISIBLE OUTCOME
- NO EFFECT ON SKIN QUALITY
- POSSIBLE SIDE EFFECTS

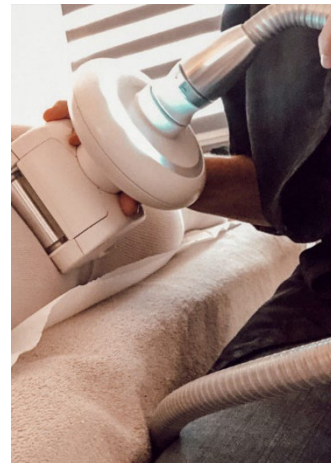

### ENDERMOLOGIE

- TREAT SMALL AND LARGE AREAS: A FULL BODY CONTOURING
- TARGET ALSO SKIN QUALITY, NOT ONLY VOLUMES
- FASTER RESULTS WITH ENDERMOLOGIE POST-TX \*\*
- ANTI-FIBROSING EFFECT WITH ENDERMOLOGIE PRE-TX \*\*\*
- FDA CLEARED FOR CELLULITE & BODY CONTOURING
- FDA CLEARED FOR LYMPHATIC & BLOOD CIRCULATION INCREASE
- OEDEMA & METABOLIC WASTES ELIMINATION
- NO SIDE-EFFECTS

\*\*Dr. Pietri (FR) - 2014 \*\*\* Plault (FR) - 2014

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + CRYOLIPOLYSIS

**INDICATIONS:** LOCALISED FAT



BEFORE

AFTER 1 CRYO 12 LPG

NUMBER OF LPG SESSIONS: 12 SESSIONS  
DURATION OF THE PROTOCOL: 10 WEEKS

**PREPARE**

**WEEK -1, -2**

LPG SESSIONS 35'

LPG 35'

**COMBINE**

**DAY OF PROCEDURE**

NO LPG TREATMENT

CRYO

**MAINTAIN**

**WEEK 1 -2-3-4**

2 LPG SESSION 35'

LPG 35' DRAINAGE

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + CRYOLIPOLYSIS

**INDICATIONS** : SKIN SAGGING



**COMBINE**

**DAY OF  
PROCEDURE**

NO LPG TREATMENT

ULTRASOUND SESSION  
5' PER AREA

**MAINTAIN**

**WEEK 1 -2-3-4-5-6**

2 LPG  
SESSIONS/WEEK 20'


ENDERMOLOGIE TWICE A  
WEEK TO FIRM AND TIGHTEN  
THE SKIN FOR 3 TO 6 WEEKS

NUMBER OF LPG SESSIONS: 6 to 12 SESSIONS  
DURATION OF THE PROTOCOL: 3 to 6 WEEKS  
MAINTENANCE ONCE A MONTH

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + CRYOLIPOLYSIS

**INDICATIONS:** CELLULITE ASSOCIATED WITH FAT AND WATER RETENTION

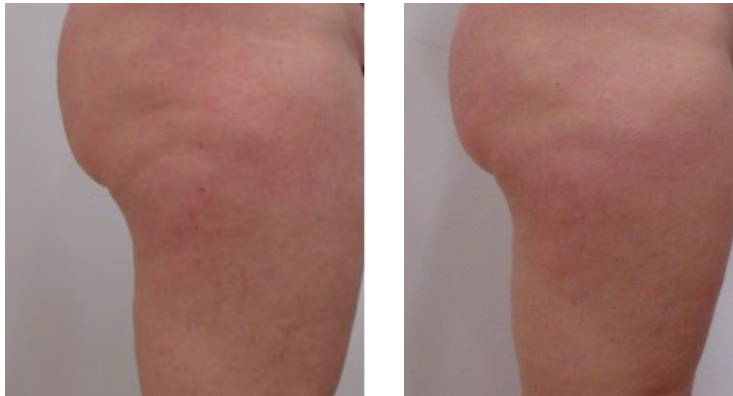

BEFORE

AFTER

Courtesy of Dr  
BLANCHEMAISON  
Paris, France

### PREPARE

#### WEEK -1 -2, 2 sessions/week

4 LPG SESSIONS 35'

LPG 35' Gynoid

### COMBINE

#### WEEK 3

CRYO

Gluteo femoral area

### MAINTAIN

#### From WEEK 4, 1 session/week

8 LPG SESSIONS 35'

LPG 35' Gynoid

NUMBER OF LPG SESSIONS: 12 SESSIONS

DURATION OF THE PROTOCOL: 3 WEEKS + 2 MONTHS

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + RADIOFREQUENCY

**INDICATIONS:** LOCALISED FAT AND LOSS OF SKIN TONICITY IN DORSAL AREA



BEFORE

AFTER

Courtesy of Dr  
BLANCHEMAISON  
Paris, France

### PREPARE

#### WEEK 1

2 LPG SESSIONS 15'

LPG 15' Pre intervention

### COMBINE

#### WEEK 2 to 5

4 RF SESSIONS

Dorsal area

### MAINTAIN

#### From WEEK 6 to 8, 2 sessions/week

6 LPG SESSIONS 35'

LPG Redensify

NUMBER OF LPG SESSIONS: 8 SESSIONS  
DURATION OF THE PROTOCOL: 2 MONTHS



# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE + LIPOSUCTION

### PRE-SURGERY

- HELP VASCULAR EXCHANGES, TO PREPARE THE TISSUE FOR SURGERY AND HEALING

### POST-SURGERY


- HELP HEALING PROCESS,
- RESTAURE THE LYMPHATIC SYSTEM
- TREAT SKIN IRREGULARITIES
- SMOOTH CELLULITE
- MINIMIZE BRUISES
- DRAIN BRUISES
- OPTIMISE FINAL RESULTS, MINIMISATION OF COMPLICATIONS AND SUBSEQUENT REVISION

# 7. COMBINED BODY PROTOCOLS

## ENDERMOLOGIE® + LIPOSUCTION

**SIDE EFFECTS**

**INDICATIONS:** OEDEMA AND BRUISES POST LIPO

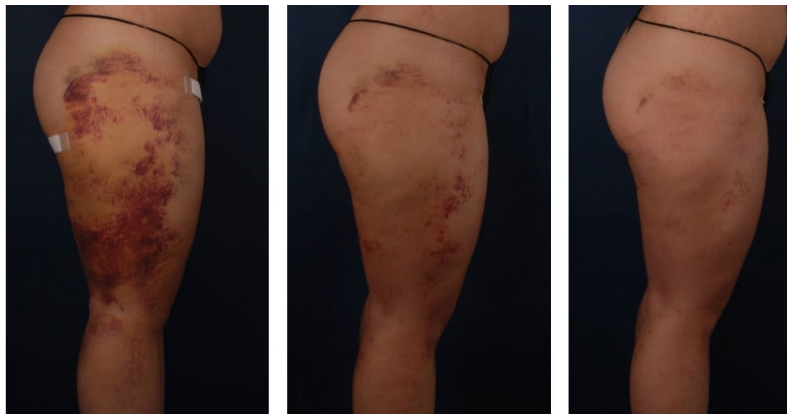

NUMBER OF LPG SESSIONS: 20 SESSIONS  
DURATION OF THE PROTOCOL: 10 WEEKS

**BEFORE SURGERY**

**PREPARE**

**WEEK -1 -2**

2 LPG SESSIONS 35'

**LPG TREATMENT:**

LPG 35'

**COMBINE**

**DAY OF PROCEDURE**

NO LPG TREATMENT

Liposuction

**AFTER SURGERY**

**MAINTAIN**

**WEEK 1 -2**

3 LPG SESSIONS 35'

LPG 35' drainage

**WEEK 3-4**


3 LPG SESSIONS 35'

LPG 35' fibrosis

**WEEK 5-8**

3 LPG SESSIONS 35'

LPG 35' cellulite

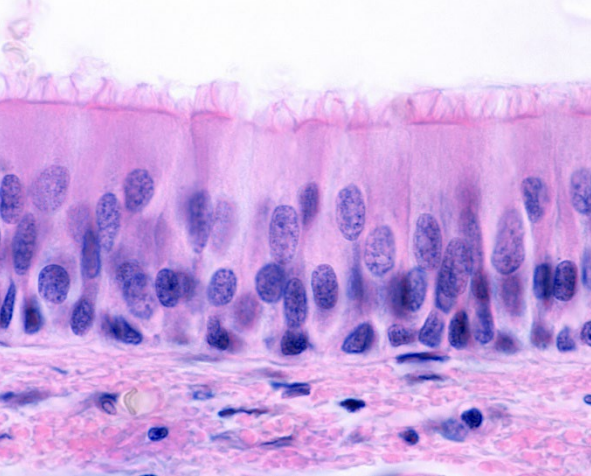

8.

COMBINED BODY RESULTS

---

# 8. COMBINED BODY RESULTS ENDERMOLOGIE® + LIPOSUCTION



BEFORE




AFTER 2 SESSIONS


COURTESY DR PULETTI, ITALY

# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + LIPOSUCTION

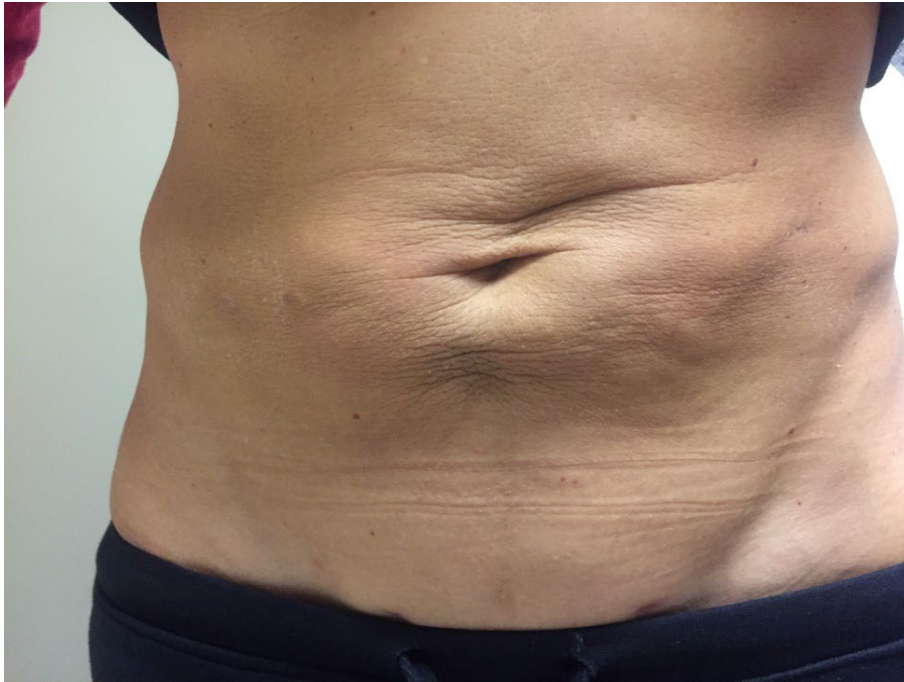

AFTER LIPO

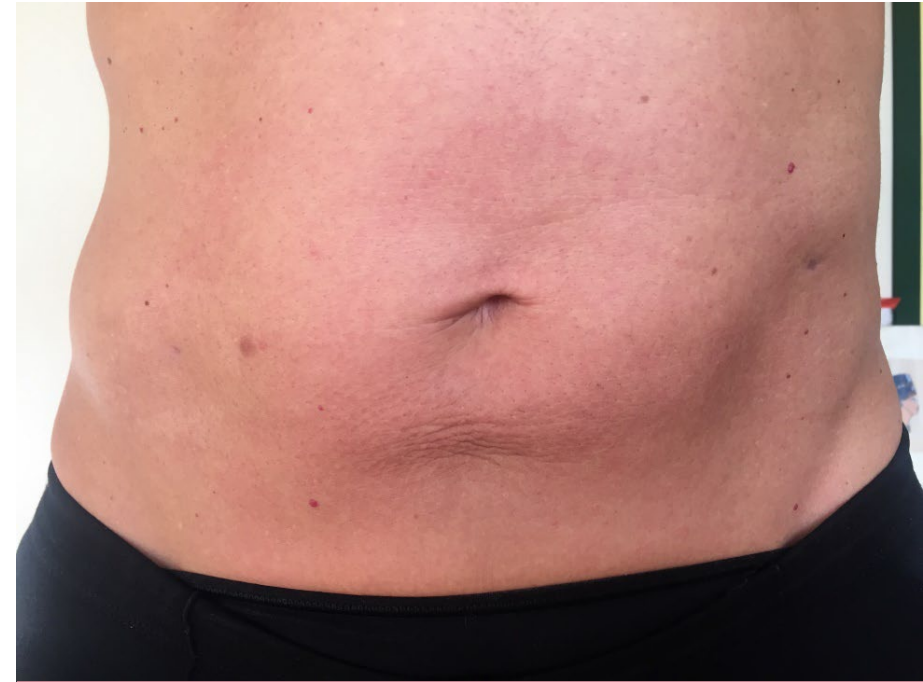

AFTER 8 LPG SESSIONS


COURTESY DR PULETTI, ITALY

# 8. COMBINED BODY RESULTS ENDERMOLOGIE® + LIPOSUCTION

POST SURGERY 1 DAY AFTER


POST SURGERY 3 DAYS AFTER

POST SURGERY 5 DAYS AFTER

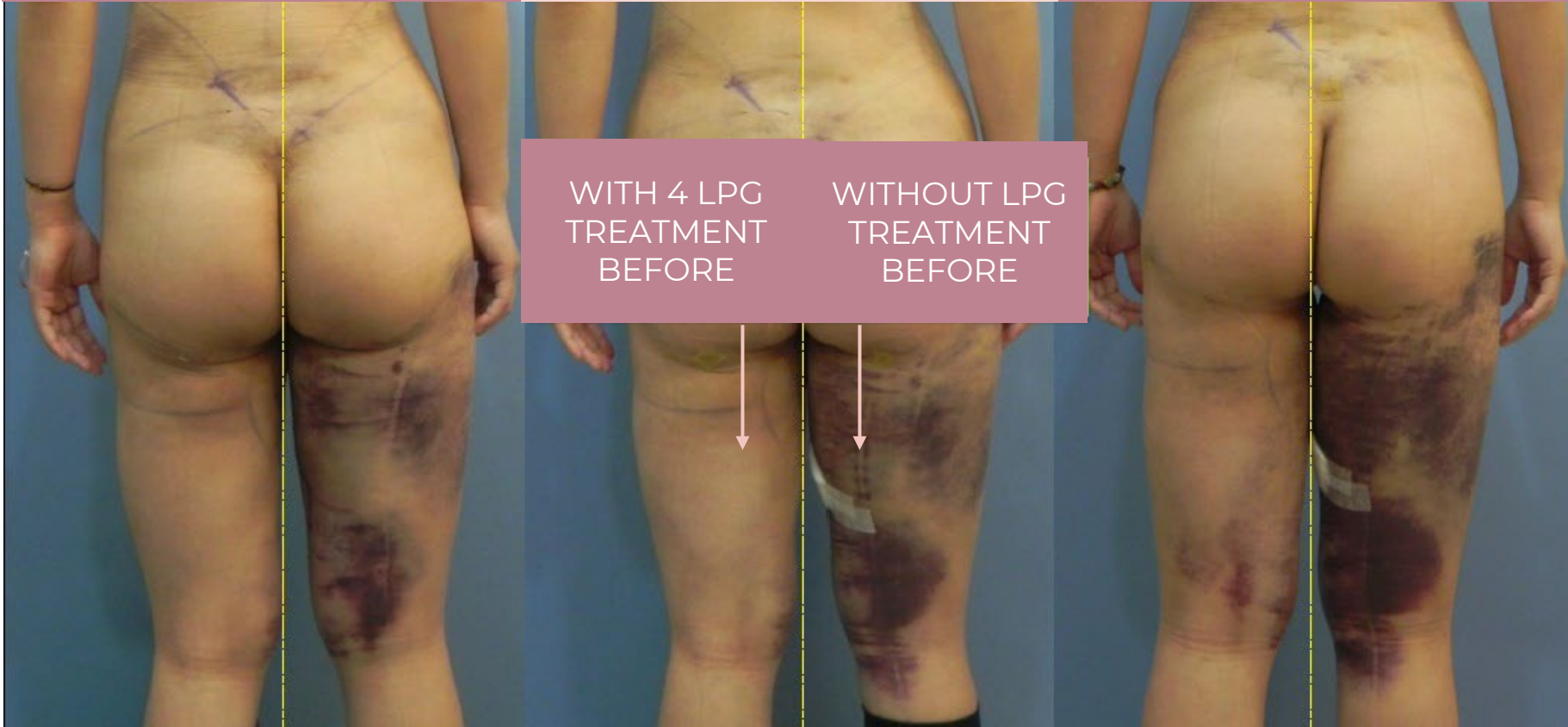


# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + LIPOSUCTION

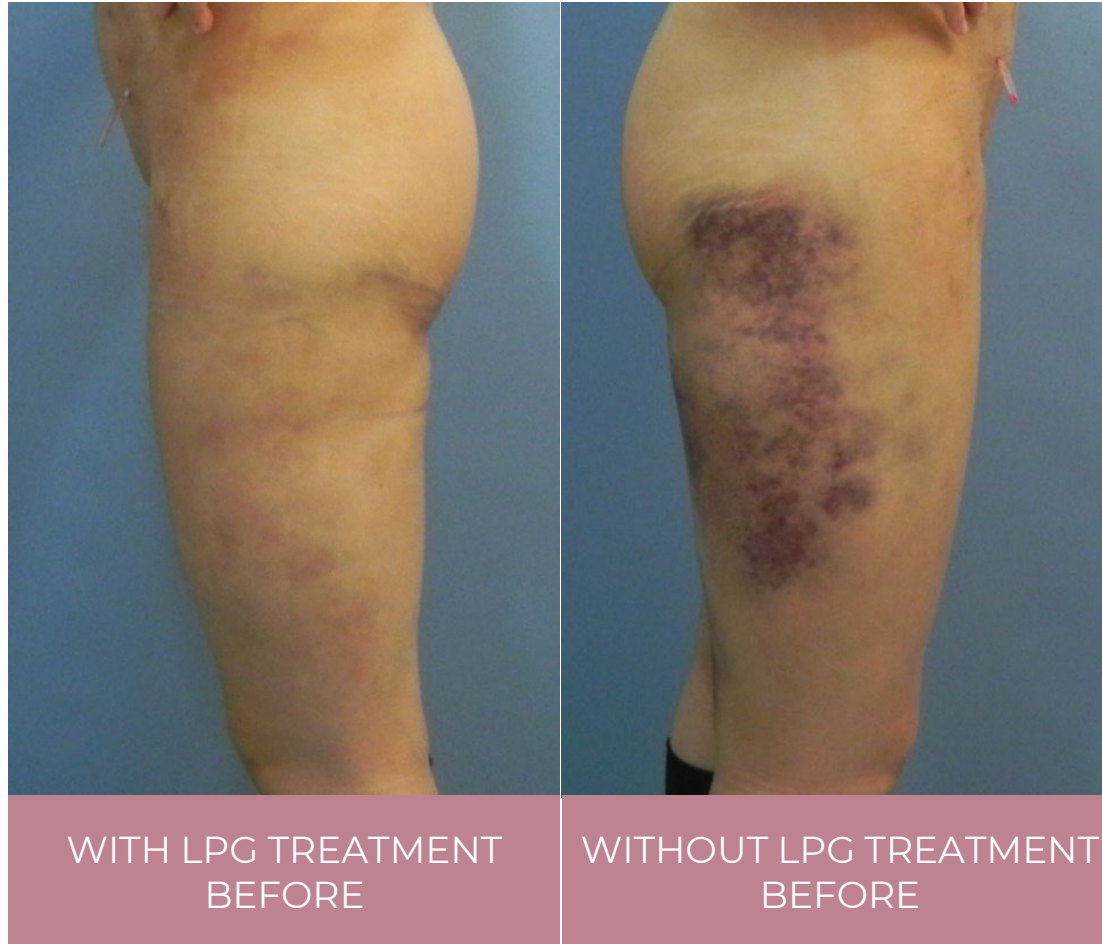

# 8. COMBINED BODY RESULTS


## ENDERMOLOGIE® + LIPOSUCTION

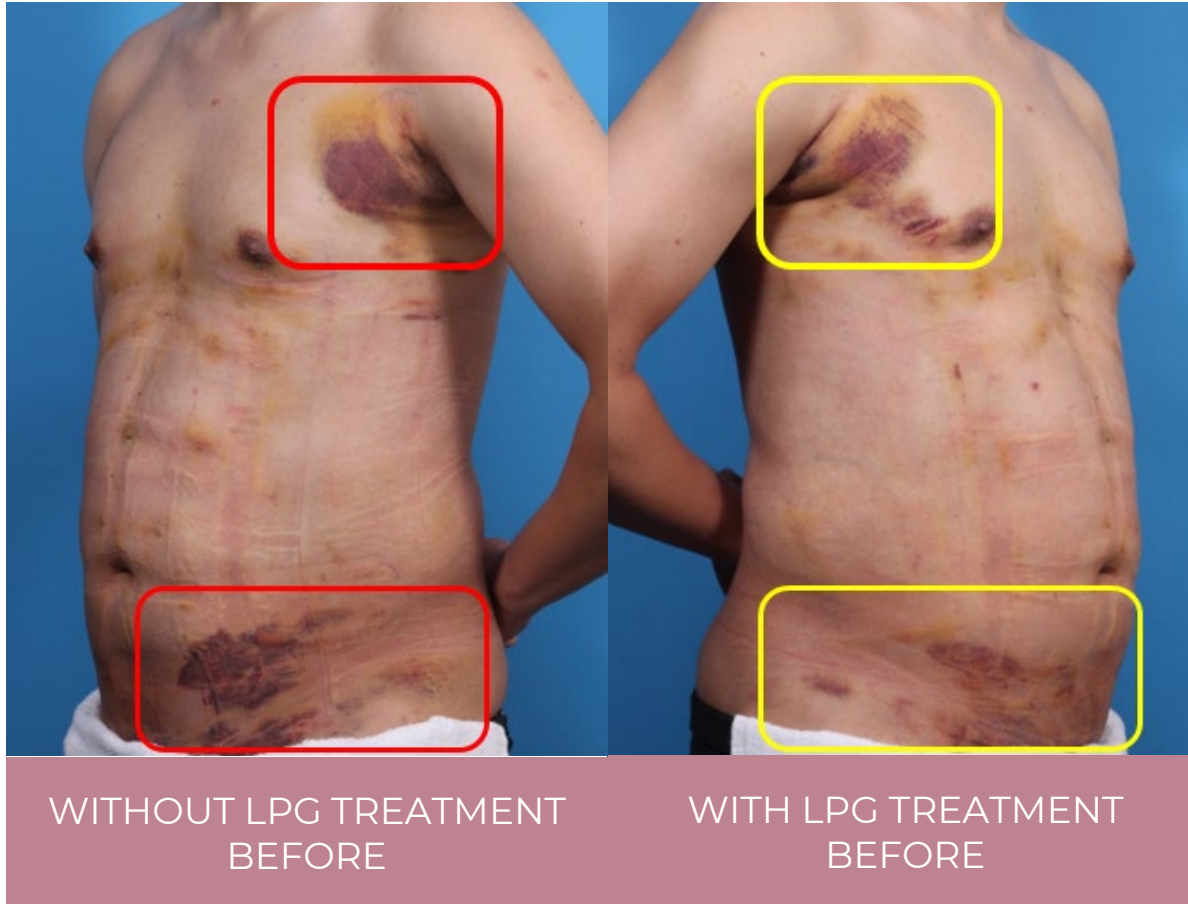



# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + LIPOSUCTION

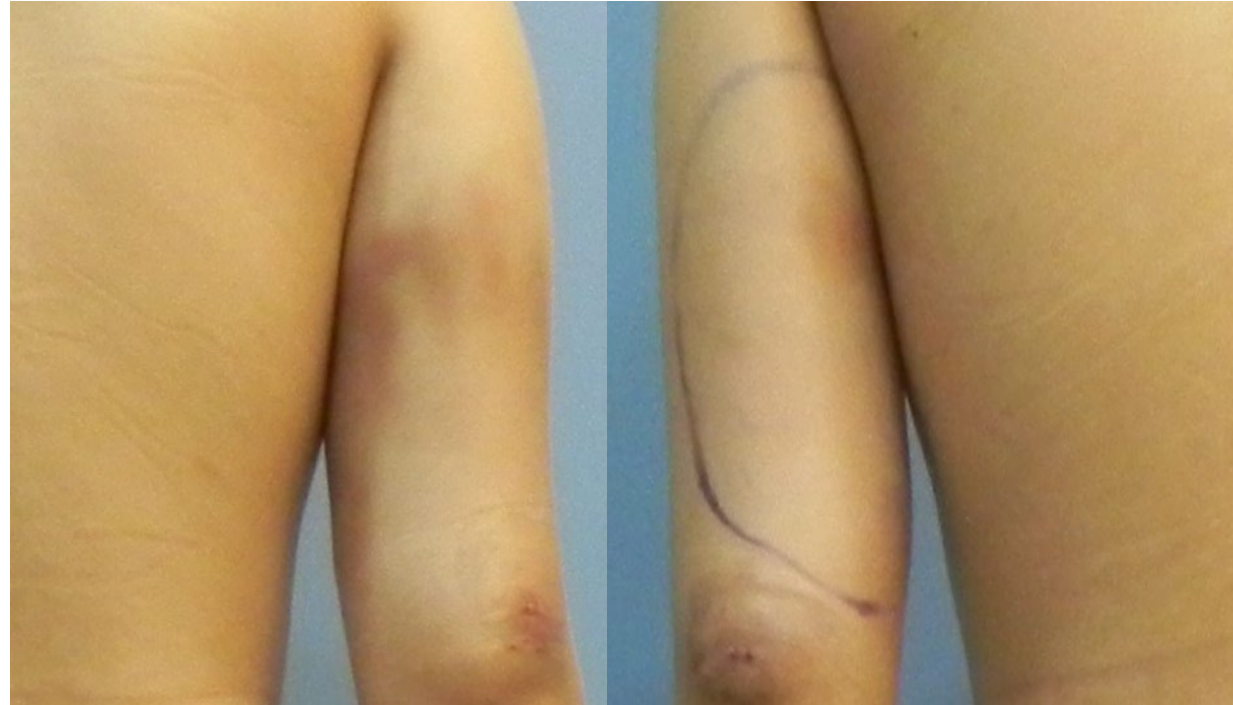

WITH LPG TREATMENT  
BEFORE

WITHOUT LPG TREATMENT  
BEFORE

# 8. COMBINED BODYRESULTS ENDERMOLOGIE® + LIPOSUCTION



COURTESY DR CUVELIER, BELGIUM

# 8. COMBINED BODYRESULTS ENDERMOLOGIE® + LIPOSUCTION




PRE-OP TREATMENT

POST-OP 1 MONTH  
4-6 LPG SESSIONS

POST-OP 3 MONTHS  
12 LPG SESSIONS

COURTESY DR CUVELIER, BELGIUM

# 8. COMBINED BODYRESULTS ENDERMOLOGIE® + LIPOSUCTION

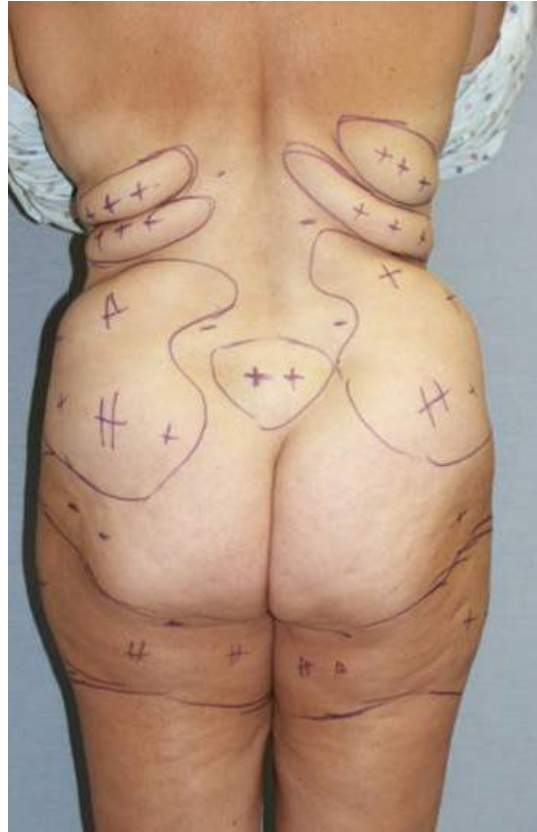

BEFORE LIPO




AFTER LIPO  
+ 10 LPG SESSIONS

COURTESY DR HURVITZ, USA

# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + MINI-INVASIVE LIPOSUCTION

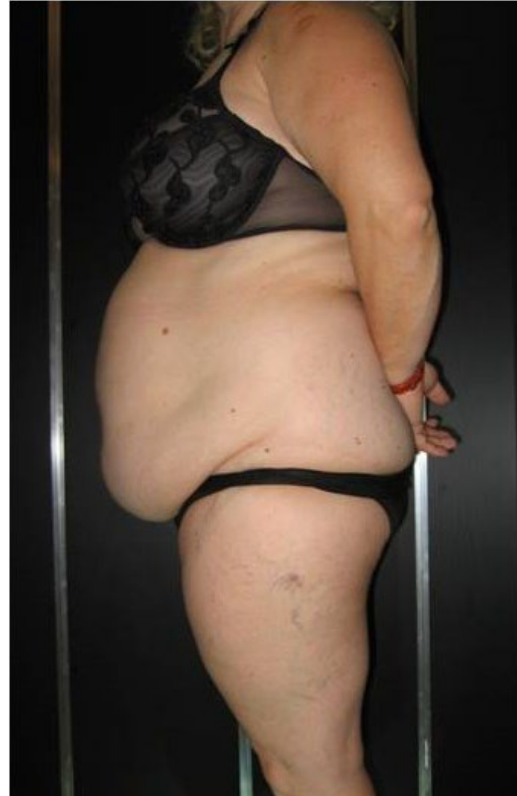

BEFORE



AFTER

COURTESY DR MAJANI, ITALY

# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + MINI-INVASIVE LIPOSUCTION



BEFORE

AFTER

COURTESY DR MAJANI, ITALY

# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + MINI-INVASIVE LIPOSUCTION



BEFORE



AFTER

COURTESY DR MAJANI, ITALY

# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + MINI-INVASIVE LIPOSUCTION



BEFORE



AFTER

COURTESY DR MAJANI, ITALY




# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + RADIOFREQUENCY



# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + RADIOFREQUENCY + CRYO

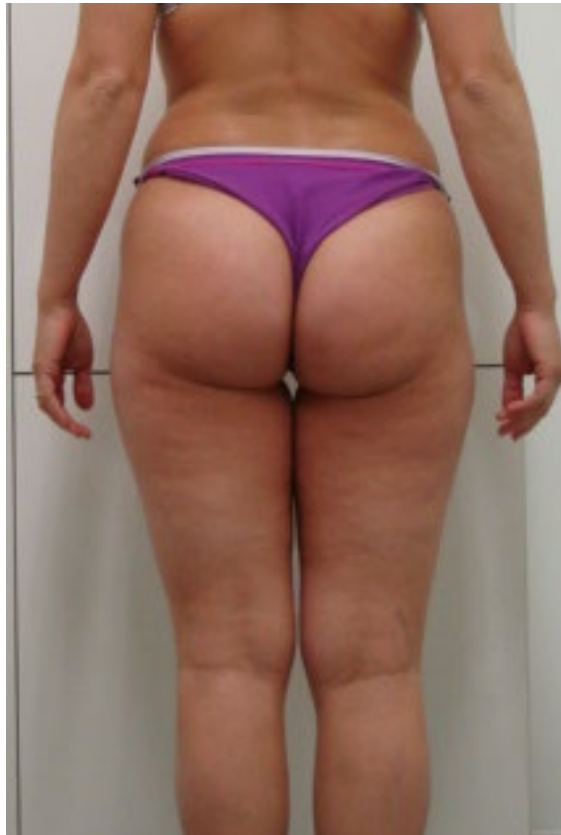

BEFORE



7 COMBINED SESSIONS


1 CRYO  
6 RF + LPG

# 8. COMBINED BODY RESULTS

## ENDERMOLOGIE® + SURGERY COMPLICATION



AFTER LIPO

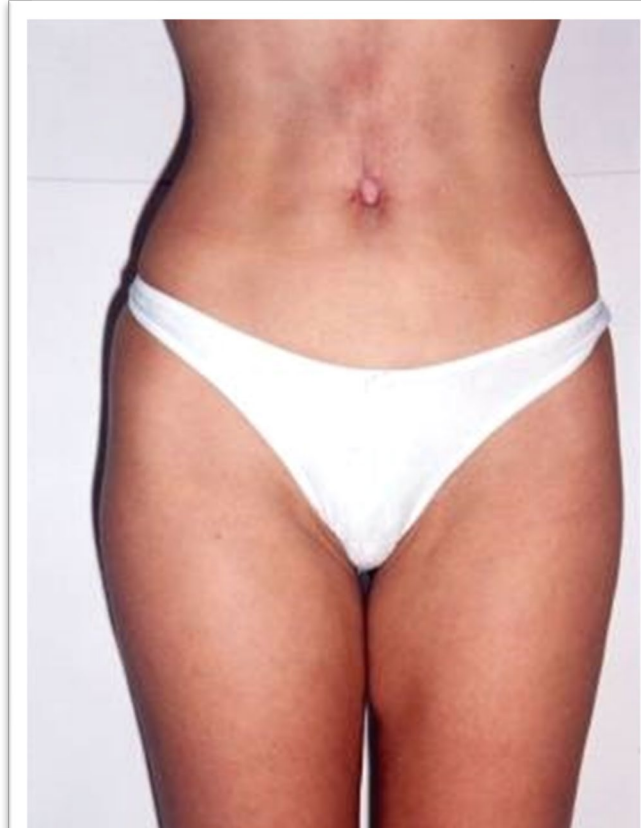

AFTER 16 SESSIONS